Transrectal Scanning Kit

REFERENCE GUIDE

CE
0120

I. Covering the Transducer

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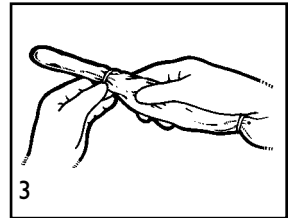
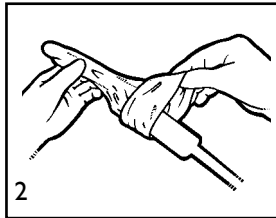
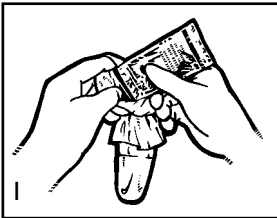
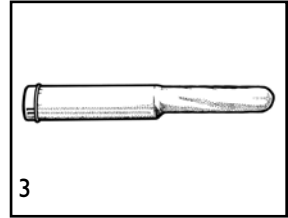
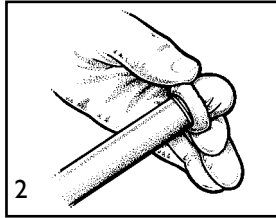
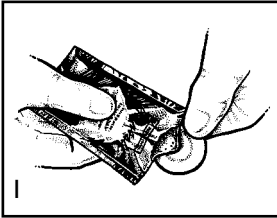
Slik brukes transducerkappen

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Cubrir el transductor

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II. Using Scanning Kit (if applicable)

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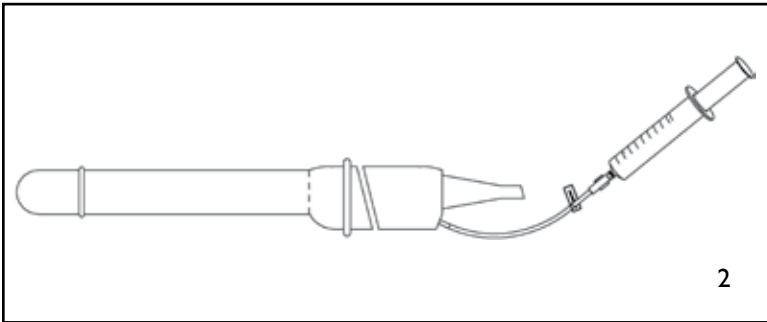
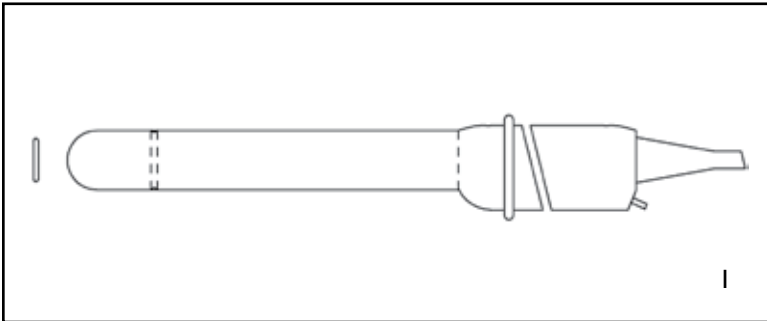
Skanningssats

INTENDED USE

- This device provides physicians with a tool for performing needle/instrument guided procedures with the use of the diagnostic ultrasound endocavity transducer.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system’s manual.*
- *Transducer covers containing natural rubber latex may cause allergic reactions. In the USA, refer to FDA Medical Alert MDA91-1.*
- *For illustration purposes only, transducer may be shown without a transducer cover. Always place a cover over transducer to protect patients and users from cross-contamination.*
- *Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm cover.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*

I. Covering the Transducer _____

1. Place an appropriate amount of gel inside cover and/or on transducer face. Poor imaging may result if no gel is used.
2. Insert transducer into cover making sure to use proper sterile technique. Pull cover tightly over transducer face to remove wrinkles and air bubbles, taking care to avoid puncturing cover.
3. Secure cover with enclosed bands.
4. Inspect cover to ensure there are no holes or tears.

II. Using Scanning Kit _____

1. Fill syringe with water and connect extension tube to syringe.
2. Press plunger down until all air has been released from tube.
3. Attach tube to transducer fill port.
4. Inject water into cover through transducer. Remove air bubbles.

III. Disposal _____

WARNING

- *Dispose of single-use components as infectious waste.*
- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *See your system’s manual for reprocessing transducer between use.*

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COPYRIGHT © 2009 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH