Reusable Acoustic Standoff Pad

REFERENCE GUIDE

CE
0120

INTENDED USE

- *The standoff is intended to control the distance (offset) between the transducer and body surfaces for optimal focusing by shifting the transducer, allowing improved visualization of near-field and superficial structures.*

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system’s manual.*
- *See your system’s manual for reprocessing transducer between use.*

I. USING ACOUSTIC STANDOFF PAD

WARNING

- *Acoustic standoff pad is packaged non-sterile and is reusable. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Do not use acoustic standoff pad if it appears damaged.*
- *Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm acoustic standoff pad.*

1. Place gel on patient to provide couplant between patient and acoustic standoff pad.
2. Apply acoustic standoff pad onto gel.
3. Place gel on pad to provide couplant between acoustic standoff pad and transducer.

II. DISPOSAL

WARNING

- *Dispose of standoff pad when damage is evident, according to regulations for biohazardous waste.*

III. CLEANING ACOUSTIC STANDOFF PAD

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *See your system’s manual for reprocessing transducer between use.*
- *Do not autoclave or gas sterilize.*

1. Clean acoustic standoff pad with either mild soap and water; 1:10 dilution of bleach-to-water or other solutions approved by your facility, not containing alcohol or any other strong, undiluted disinfectants.
2. Wipe down to remove any residue.
3. Inspect acoustic standoff pad for damage, such as cracks or breakage. If damage is evident, discontinue use.

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COPYRIGHT © 2010 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH