


Transducer Storage Rack

REFERENCE GUIDE




List of Components:

GE Healthcare, Philips, Siemens

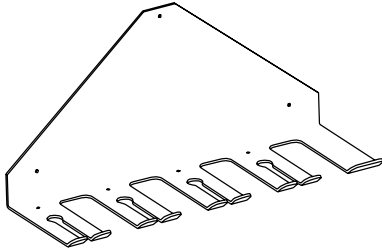

Hitachi

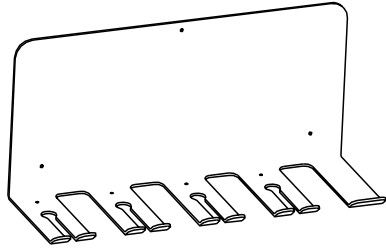

Transducer Storage Rack (1)



Transducer Stabilizer (1)



E-Z Anchor Molly Bolts (3)



Flat washers, nylon, #10 (3)



Phillips mounting screws 8/32 x 1/2 (3)

Transducer Storage Rack



English

WARNING

- To prevent electrical or wall damage, it is recommended the transducer storage rack be installed by the facilities department.
- Before attaching, you should have a phillips screwdriver and level.

Assembly

1. Place transducer storage rack against wall and level.
2. Mark the location of the three mounting holes.
3. Secure E-Z anchor molly bolts to wall with phillips screwdriver.
4. Mount transducer storage rack to molly bolts using phillips screws and washers.
5. Install rack probe support into desired location.

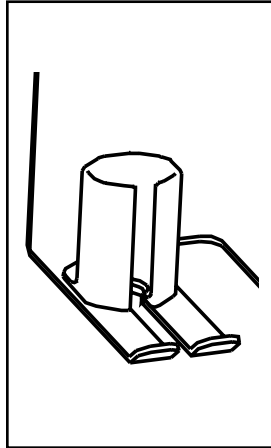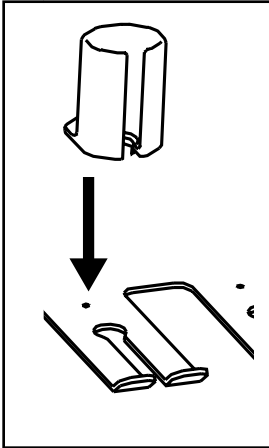

NOTE

- Contact CIVCO to order additional transducer stabilizers (610-957).
Phone: 800.445.6741 • www.civco.com.



Corporate Headquarters: 102 First Street South, Kalona, IA 52247-9589 USA
319.656.4447, 800.445.6741 | fax: 319.656.4451, 877.329.2482
www.civco.com

COPYRIGHT © 2009 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.