

# Transducer Storage Rack


---

REFERENCE GUIDE




List of Components:


GE Healthcare, Philips, Siemens

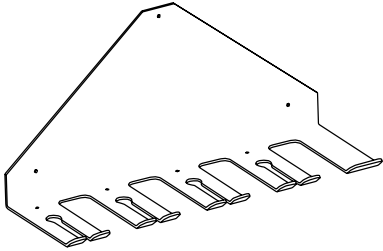

Hitachi Aloka Medical

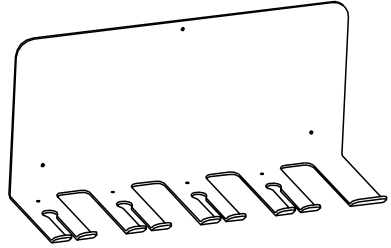

Philips (Compact Connector)

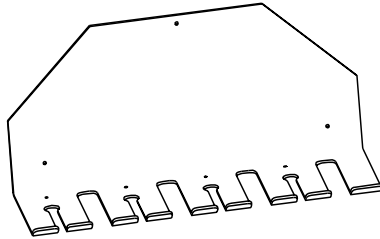

Transducer Storage Rack (1)



Transducer Stabilizer (1)



E-Z Anchor Molly Bolts (3)



Flat washers, nylon, #10 (3)





Phillips mounting screws 8/32 x 1/2 (3)

**WARNING**

- To prevent electrical or wall damage, it is recommended the transducer storage rack be installed by the facilities department.
- Before attaching, you should have a phillips screwdriver and level.

**Assembly**

1. Place transducer storage rack against wall and level.
2. Mark the location of the three mounting holes.
3. Secure E-Z anchor molly bolts to wall with phillips screwdriver.
4. Mount transducer storage rack to molly bolts using phillips screws and washers.
5. Install rack probe support into desired location.

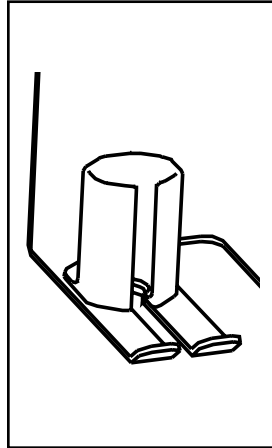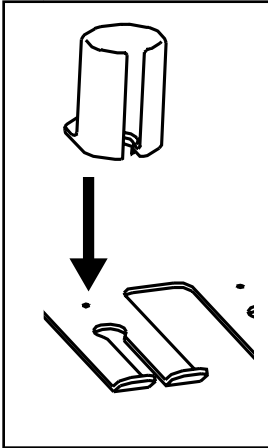

**NOTE**

- Contact CIVCO to order additional transducer stabilizers (610-957).  
Phone: 800.445.6741 • [www.civco.com](http://www.civco.com).

**Reprocessing**

**WARNING**

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.
1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.



---

**Corporate Headquarters:** 102 First Street South, Kalona, IA 52247-9589 USA  
319.656.4447, 800.445.6741 | fax: 319.656.4451, 877.329.2482  
[www.civco.com](http://www.civco.com)

COPYRIGHT © 2011 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.

043-566F