



Laparoscopic Transducer Cover with Latex-Free Tip

Reference Guide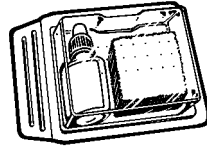

CIVCO

CE
0120

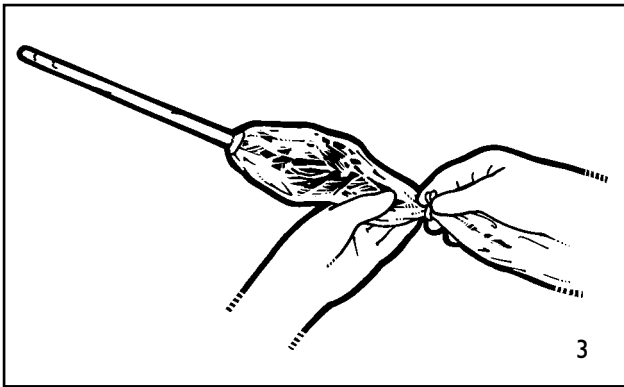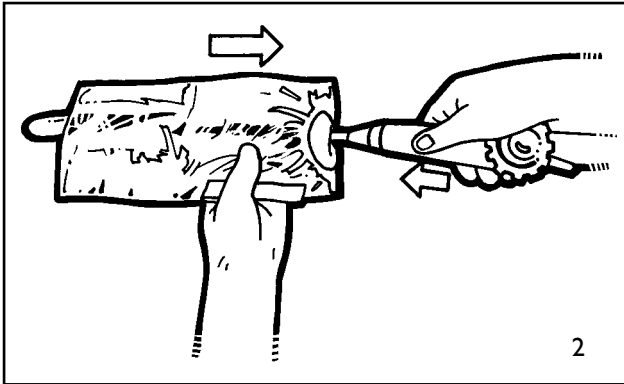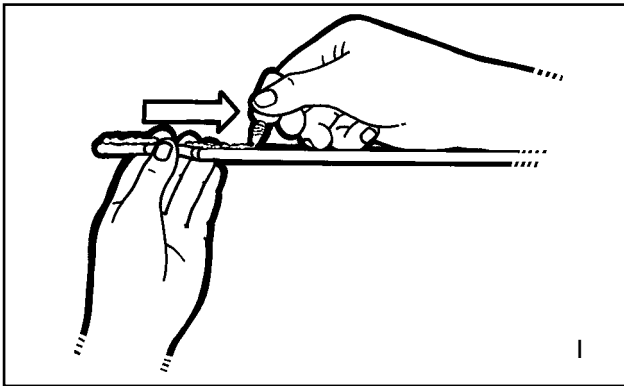INTENDED USE

- The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Dispose of single-use components as infectious waste.*
- *See your system's manual for reprocessing transducer between use.*
- *Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm cover.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*

Covering the Transducer

WARNING

- *Inspect cover to ensure there are no holes or tears.*
- *Lubricate probe shaft with Endo-Lube™ to aid in application and removal of cover. Place appropriate amount of saline solution inside cover. Poor imaging may result if no coupling agent is used.*

Using Transducer with Trocar

- Cover can be used on 10mm transducers only. To accommodate use of transducer cover, a trocar 2mm larger than transducer size is recommended for use.
- Lubricate exterior of latex cover and trocar seal, as indicated in Endo-Lube instructions.
- To avoid tearing cover, transducer must be inserted or removed in non-angled position into trocar.
- Maintain taut fit of cover on transducer.

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COPYRIGHT © 2011. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH