Table and Rail Adapters

REFERENCE GUIDE

CE

INTENDED USE

- The table adapter is intended to attach to a flat or round surface and provide an interface to accept CIVCO Assist attachments.

WARNING

- *Ensure device is stable before use.*

Using Table Adapter

1. If attaching to flat surface, insert “V” block prior to use. If attaching to a round or irregular surface, remove “V” block prior to use.
2. Attach table adapter by tightening handle in a clockwise motion.
3. Ensure table adapter and any device installed into attachment point is secure.

Cleaning and Disinfecting




WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*

- *Do not place in mechanical washer.*

- *Do not autoclave to sterilize.*

1. After each use, discard single-use, disposable components.
2. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.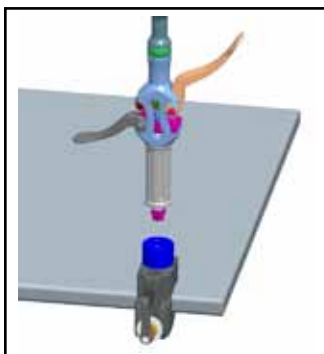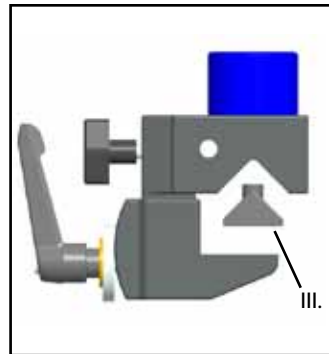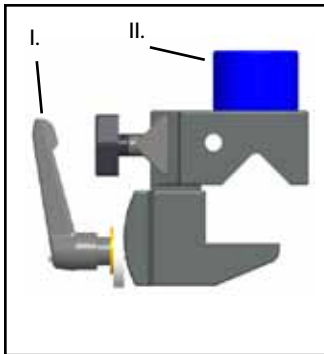

- I. Handle
- II. Attachment Points
- III. “V” Block

INTENDED USE

- Rail adapter is intended to attach to common rails and provide an interface to accept CIVCO Assist attachments.

WARNING

- *Ensure device is stable before use.*

Using Rail Adapter


1. Attach rail adapter by tightening knob in a clockwise motion.
2. Ensure rail adapter and any device installed into attachment point is secure.

Cleaning and Disinfecting

WARNING

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *Do not place in mechanical washer.*
- *Do not autoclave to sterilize.*

1. After each use, discard single-use, disposable components.
2. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

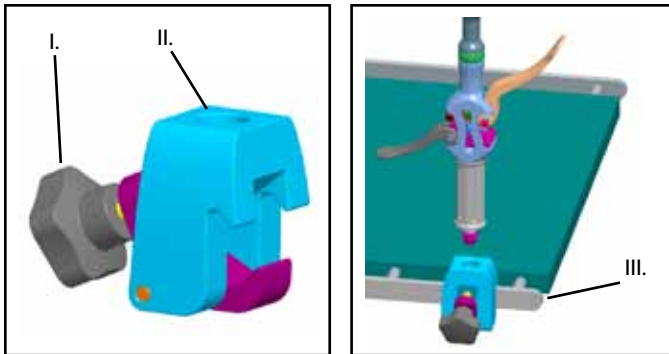- I. Knob
- II. Attachment Points
- III. Rail

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COPYRIGHT © 2011. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA.
MPS GmbH