# Percutaneous Localizer

---

REFERENCE GUIDE

**CIVCO** *Assist*<sup>™</sup>

CE

**INTENDED USE**

- Percutaneous localizer is intended to be used as a skin surface marker in a CT image.

**WARNING**

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Percutaneous localizer is packaged non-sterile and is reusable. To avoid possible patient contamination, ensure percutaneous localizer is properly cleaned, sterilized or disinfected before each use.*
- *Percutaneous localizer may be used up to 10 times as long as it does not appear damaged or triangular area is not bent.*
- *Avoid bending triangular area.*

**USING PERCUTANEOUS LOCALIZER**

1. Place percutaneous localizer onto patient over target area with longest edge parallel to imaging plane. Secure with tape (*not provided*).
2. CT image appears as series of dots corresponding to narrow bars; short lines at each end correspond to angled numbered edge.
3. Locating entry point.
  - Match dots on CT image with narrow bars to determine horizontal location on grid.
  - Measure gap in centimeters between last dot and inside edge of short line on CT image.
  - Match numbers along angled numbered edge to determine vertical location on grid.

**REPROCESSING**

**WARNING**

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *The recommended protocol for reprocessing the items listed above has been evaluated for effectiveness and compatibility. If choosing to use a reprocessing protocol other than those listed, a validation must first be completed to ensure effectiveness and compatibility.*
- *Inspect for damage such as cracks or breakage. If damage is evident, discontinue use.*


**CLEANING**

1. Remove visible contaminants using a small, soft instrument brush. Keep from drying out until complete cleaning can be accomplished.
2. Soak for minimum of five minutes in neutral pH, low foaming enzymatic detergent, such as ENZOL® Enzymatic Detergent (Johnson & Johnson). While immersed, use instrument brush to remove trapped contaminants from surfaces and features. If visible contaminants cannot be easily removed, repeat soaking procedure for an additional five minutes. Remove from cleaning solution and remove any remaining residue with dry wipe. Follow cleaning solution manufacturer’s directions for use and recommendations for concentration.

**DISINFECTING AND STERILIZING**

- Sterilize using steam (autoclave) at 132-138°C for 15 to 30 minutes.
- High-level disinfect using CIDEX OPA® ortho-Phthalaldehyde Solution (Johnson & Johnson) or equivalent .55% ortho-phthalaldehyde-based solution. Follow manufacturer’s instructions and recommendations for concentration, time of contact and post-process procedure.

- High-level disinfect or sterilize using CIDEX® or CIDEX® Plus Activated Dialdehyde Solution (Johnson & Johnson) or equivalent glutaraldehyde-based solution. Follow manufacturer's instructions and recommendations for concentration, time of contact and post-process procedure.
- Inspect for damage such as cracks or breakage. If damage is evident, discontinue use.

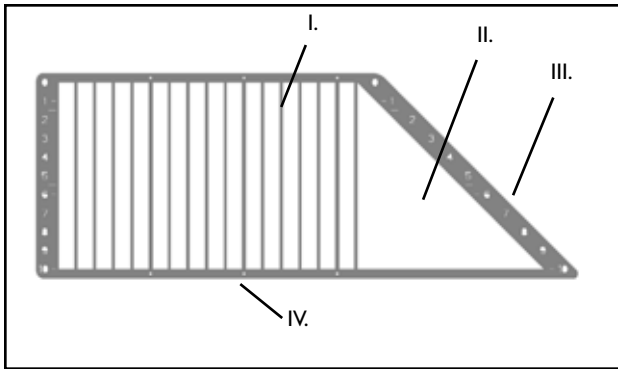- I. Narrow Bars (corresponds with dots on CT image)
- II. Triangular Area
- III. Angled Numbered Edge (corresponds with short lines on CT image)
- IV. Long Edge

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COPYRIGHT © 2009. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH