

ENDOCAVITY / GENERAL PURPOSE PROBE BAG

REFERENCE GUIDE



ITEMS COVERED BY THIS INSTRUCTION

Part Number	Description	Quantity
610-2417	Endocavity Ultrasound Probe Bag	800 Bags
610-2418	General Purpose Ultrasound Probe Bag	800 Bags
610-2442	"Clean and Disinfected" Label	1000 Labels
610-689	Bands (Orange)	100 Bands
610-690	Bands (Green)	500 Bands
610-2437	"Biohazard" Label	500 Labels
610-1327	CIVCO Ultrasound Probe Storage Cabinet; Includes U.S. power cord	1 Unit
610-2420	CIVCO Ultrasound Probe Storage Cabinet; Cord sold separately - Not available for U.S. or Canada	1 Unit

PRODUCT DESCRIPTION

Product is designed for transport and storage of ultrasound transducers.

WARNING

- *Product is non-sterile and is not intended for patient use.*
- *Dispose of single-use components as infectious waste.*

TRANSPORTATION

1. Transport clean probe in clean bag to exam room.
 - A "Clean and Disinfected" label may be used to indicate clean status.
 - A label or band may be used to close bag.
2. Post-exam, place dirty probe in same bag or use a new, clean bag.
 - A "Biohazard" label may be used to indicate dirty status.
 - A label or band may be used to close bag.
3. Transport probe to reprocessing location and dispose of bag according to facility guidelines.
4. Post-reprocessing, use new, clean bag to transport disinfected probe to storage location.
 - A "Clean and Disinfected" label may be used to indicate clean status.
 - A label or band may be used to close bag.

STORAGE

1. Place clean probe in ventilated storage cabinet, such as CIVCO's Ultrasound Probe Storage Cabinet.

NOTE: ■ If using CIVCO's Ultrasound Storage Cabinet, a bag is not required for use during probe storage.

2. If ventilated storage cabinet is not available, a new, clean bag may be used for storage between each procedure to avoid re-contamination of the probe.



CIVCO Medical Instruments Co., Inc.
102 First Street South, Kalona, IA 52247-9589
319.248.6757, 800.445.6741
Fax: 319.248.6660, 877.329.2482

COPYRIGHT © 2021 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.

WWW.CIVCO.COM

2021-07-27 043-1018A