Cord Covers & System Drapes

CIV-Flex™ • Poly

REFERENCE GUIDE

CE
0120

INDICATIONS FOR USE / INTENDED USE

- Protective cover or sheath placed over equipment. The cover helps prevent transfer of microorganisms, body fluids, and particulate material to the patient and healthcare worker. The cover also provides a means for maintenance of a sterile field (sterile covers only). CIVCO Poly Covers are furnished sterile; single use patient / procedure, disposable.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Disposable covers and drapes are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Dispose of single-use components as infectious waste.*
- *Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.*
- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *See your system's manual for reprocessing transducer between use.*

COVERING THE CORD _____

1. Using proper sterile technique, hold cord cover so entire length does not unfold, possibly causing contamination.
2. Insert distal end of cord into cover and proceed toward proximal connection, sliding cover to desired position.
3. Secure cover with enclosed bands.
4. Inspect cover to ensure there are no holes or tears.

COVERING THE SYSTEM _____

1. Using proper sterile technique, hold system drape so entire length does not unfold, possibly causing contamination.
2. Remove backing from adhesive strips and secure drape to control panel.
3. Inspect drape to ensure there are no holes or tears.

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COPYRIGHT © 2015 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH