Connector Cover

REFERENCE GUIDE

CE
0120

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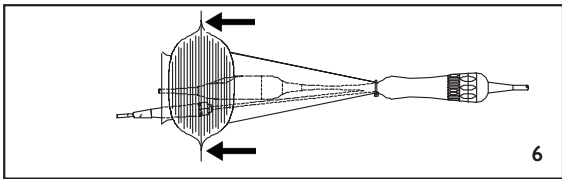
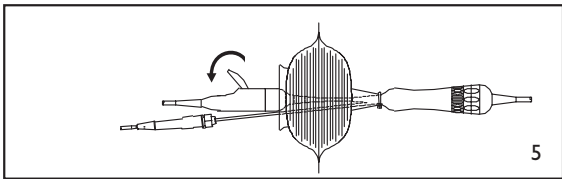
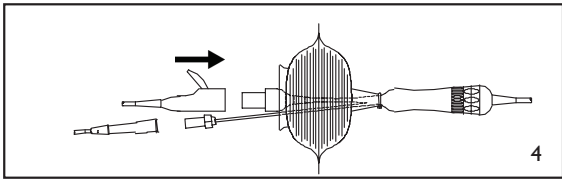
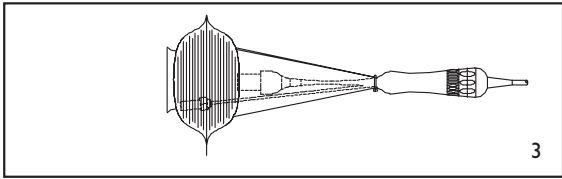
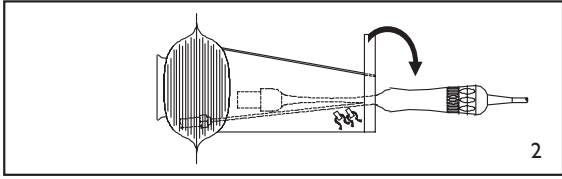
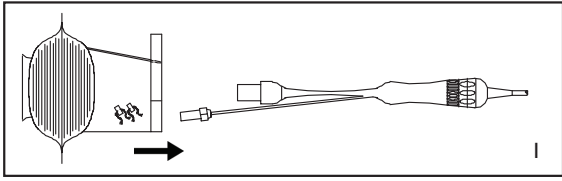
INTENDED USE

- The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your catheter, see your system's manual.*
- *Insert catheter into cover making sure to use proper sterile technique. See illustrations.*
- *Inspect cover to ensure there are no holes or tears.*
- *Dispose of single-use components as infectious waste.*
- *See your system's manual for reprocessing cable between uses.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.*

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COPYRIGHT © 2012. ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH