# TEE TRANSDUCER HOLDER

---

REFERENCE GUIDE

## INTENDED USE

TEE transducer holder allows positioning and holds a TEE transducer during procedures.

## CAUTION

*Federal (United States) law restricts this device to sale by or on the order of a physician.*

## WARNING


- Verify TEE transducer fits securely in TEE transducer holder.
- Do not use if device appears damaged.

## USING TEE TRANSUDCER HOLDER

1. Attach TEE transducer holder by turning base fully into attachment point and secure with supplied wrench. Do not over tighten.
2. Loosen single-point locking mechanism to orient TEE transducer holder to desired position. After final position is achieved, tighten single-point locking mechanism to secure.

- I. Single-Point Locking Mechanism
- II. Base
- III. Attachment Points
- IV. Radiolucent Stretcher
- V. Table Adapter
- VI. Rail Adapter

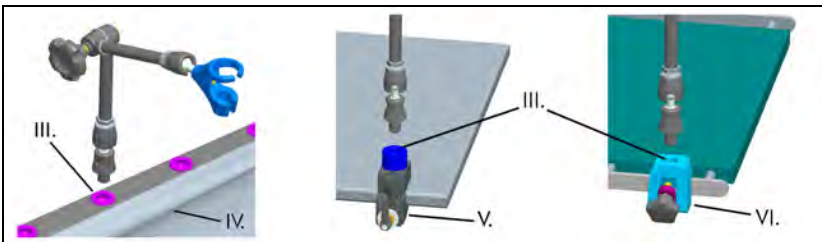## CLEANING AND DISINFECTING

### WARNING

- Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.
- Do not place in mechanical washer.
- Do not autoclave to sterilize.

1. Wipe all surfaces with common germicidal or antiseptic wipe, such as alcohol.

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COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
CIVCO Medical Instruments Co., Inc.