

# TRANSPORTATION STAND

*For use with Single-sided and Dual-sided Table Mount*

---


REFERENCE GUIDE






**ASSEMBLING TRANSPORTATION STAND**

1. Insert threaded stud end of tube through center of rolling base.
2. Slide locating washer over threaded stud and nest into center of hole in rolling base.
3. Turn locking knob onto threaded stud and secure firmly.

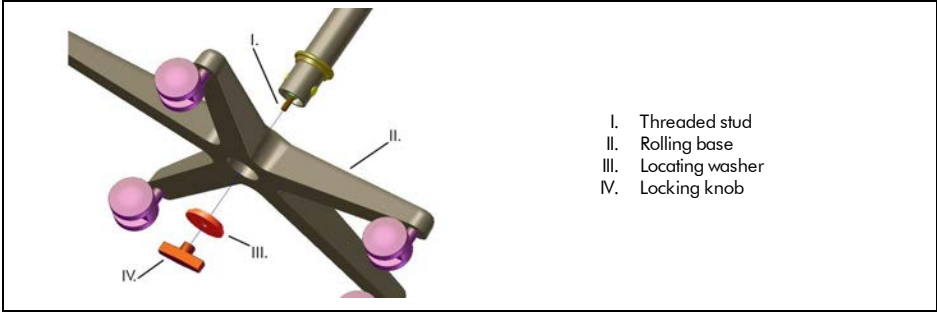

4. The transportation stand provides a convenient and mobile platform to allow ease of cleaning and handling within the surgical environment. Stand shown with stabilizer and table mount clamped in place.

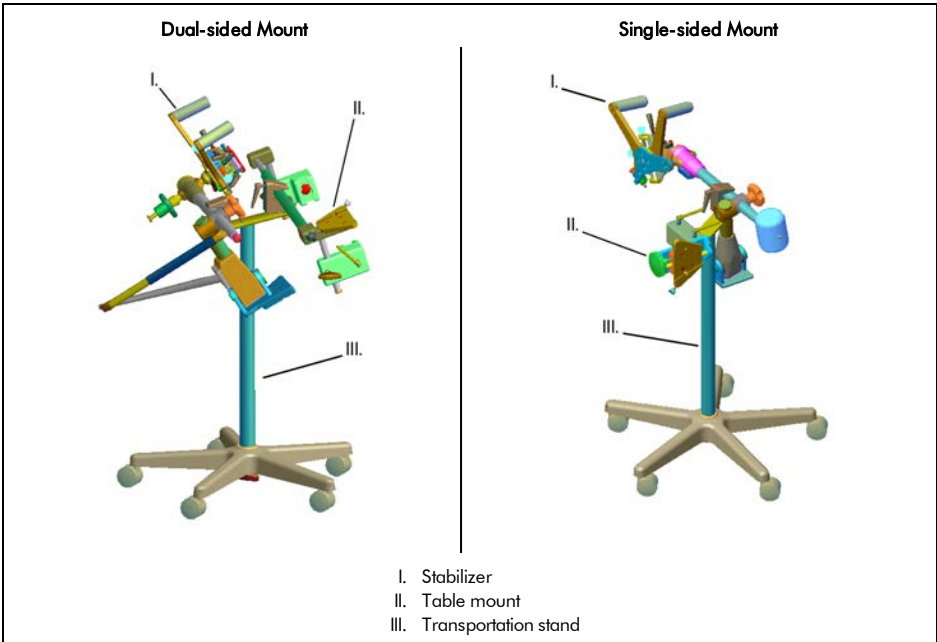



CIVCO Medical Instruments Co., Inc.  
102 First Street South, Kalona, IA 52247-9589  
319.248.6757, 800.445.6741  
Fax: 319.248.6660, 877.329.2482



---

COPYRIGHT © 2020 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.

[WWW.CIVCO.COM](http://WWW.CIVCO.COM)

2020-05-04 043-763D