

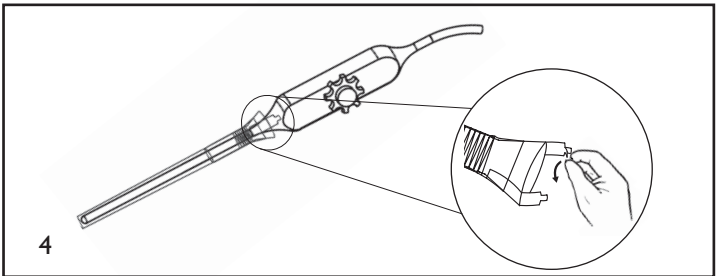
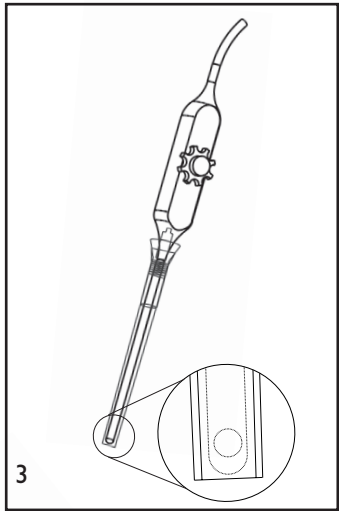
Latex-Free Transesophageal Echocardiography (TEE) Transducer Cover

REFERENCE GUIDE

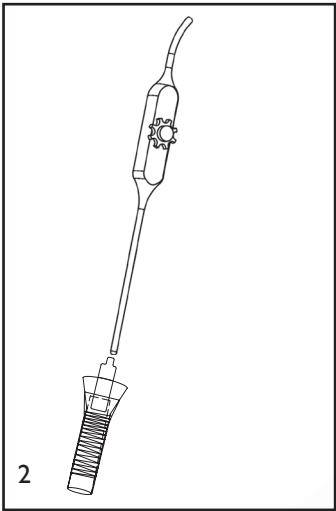CE
0120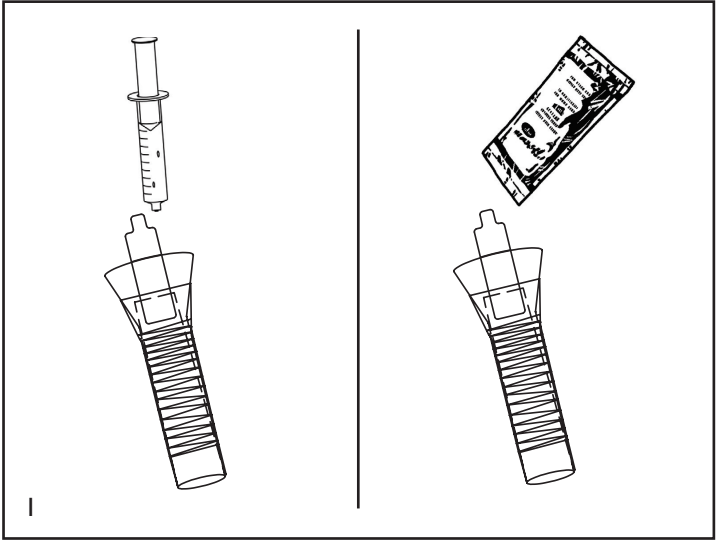

INTENDED USE

- The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm cover.*
- *Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.*

Covering the Transducer

1. Place an appropriate amount of gel inside cover and/or on transducer face. Poor imaging may result if no gel is used.
2. Carefully insert transducer into cover. Pull cover tightly over transducer face to remove wrinkles and air bubbles, taking care to avoid puncturing cover. Position cover so seams do not interfere with lens.
3. Inspect cover to ensure there are no holes or tears.
4. Remove tape liners and secure applicator to the transducer.

NOTE

- Extra adhesive tape is included to secure excess cover material to applicator.

Disposal

WARNING

- *Dispose of single-use components as infectious waste.*
- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow infection control policies established by your facility.*
- *See your system's manual for reprocessing transducer between use.*

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COPYRIGHT © 2013 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
MPS GmbH