

2.00"

PCi
MEDICAL

800-862-3394

Precautionary Statement

Dust may irritate eyes. Repeated exposure to skin may cause itching. If irritation occurs flush eyes with water and wash off skin in water or shower. Read MSDS provided.

Active Ingredient:
Sodium Carbonate (Na₂CO₃)

OXID OUT Neutralizer NO-QT

An environmentally friendly neutralizer for Revital-Ox Resert XL HLD™.

DOSE

1 bottle neutralizes up to 1.25 Quarts of Resert in 60 seconds.

INSTRUCTIONS FOR USE

- STEP 1:** Remove cap and fill bottle with water.
- STEP 2:** Replace cap and shake well.
- STEP 3:** Pour contents into a 2 liter open bowl.
- STEP 4:** Pour up to 1.25 liters of Resert into bowl. Some foaming will occur.

WARNING: Do **NOT** pour the OXID OUT directly into a GUS Container, this will cause the container to overflow.

Re-Order # NO-QT

NET WT: .63 OZ (18 Grams)

1305 REV C

4.00"