

Glute-Out[®] Neutralizer NG-5GAL

An environmentally friendly neutralizer for OPA & Glutaraldehyde

DOSE

One bottle neutralizes **5 gallons (19 L)** of OPA or Glutaraldehyde in 5 minutes.

NOTE: A dramatic color change will occur during neutralization.

Instructions for Use

- DO NOT put Glute-Out[®] into an automated reprocessor. Used disinfectant should be transferred into a separate tank for neutralization.
- When used in an open soak bowl, sprinkle evenly into solution and stir until all powder is dissolved.
- After neutralization, dispose to drain while running cold water. Thoroughly rinse container before pouring fresh disinfectant. Read MSDS.

Re-Order # NG-5GAL

Precautionary Statement

Dust may irritate eyes. Repeated exposure to skin may cause itching. If irritation occurs flush eyes with water and wash off skin in water or shower.

Active Ingredient:
GLYCINE

PCi⁺
MEDICAL

800-862-3394

Northeast Silk Screen
O.Berk
PCi 5 Gal
5-16-12
10x16 305