

**PCi**<sup>+</sup>  
**MEDICAL**  
800-862-3394

Re-Order #  
NG-QT

### Glute-Out<sup>®</sup> Neutralizer NG-QT

DOSE	INSTRUCTIONS FOR USE
1 bottle neutralizes 1 quart (1 L) of OPA or Glutaraldehyde in 5 minutes.	Sprinkle into Glute/OPA. After 5 minutes dispose to drain while running cold water.

**NOTE:** Dramatic color change will occur during neutralization.  
Read MSDS. Active Ingredient: Glycine

Northeast Silk Screen  
O.Berk  
PCi\_NG-QT  
5-3-12  
10 x 16 305n