




Storage Rack for Small Scopes

Included:

- Qty. (1) Scope Holder Shelf
- Qty. (3) 'Bottomless' Tubes
- Qty. (4) EZ Anchor Wall Anchors and Screws

Wall Mounting:

1. You will need the following items before attempting to install this unit.
 - #2 Phillips Screwdriver
 - Pencil
 - Level
 - Measuring tape
2. Determine the general location for the Storage Rack.
3. Measure up from the floor to the recommended height of 48".
4. Using a level, draw a 12" line. This line will be the bottom edge of the shelf.
5. Hold the bottom edge of the shelf along the line. Using a pencil mark the (4) locations for the wall anchors by drawing a dot using the (4) mounting holes as a guide.

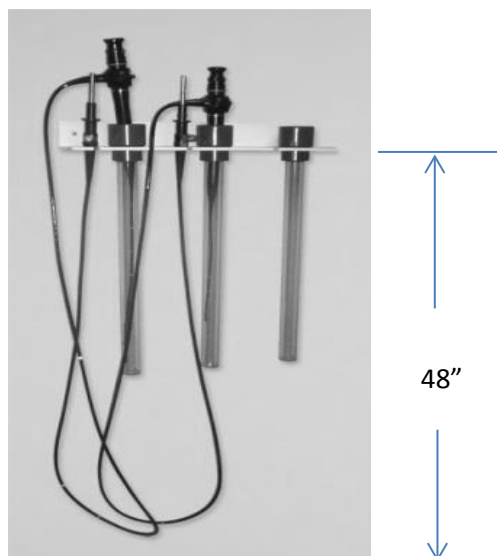

6. Place the shelf aside. Using the #2 screwdriver, screw the wall anchors into the (4) marked locations.
7. Place the Scope Holder Shelf back over the installed anchors and attach shelf to wall with the (4) Philips head screws supplied. Tighten until secure.
8. Insert the bottomless tubes in the rack as shown.
9. Store clean, dry scopes as shown above.

Cleaning:

The bottomless tubes and shelf can be washed with a mild, non-abrasive detergent and rinsed in potable water or in keeping with your facility's Infection Control guidelines.