

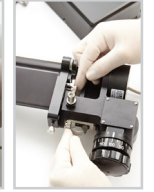


Reprocessing CIVCO Steppers & Stabilizers

Including EX³™, Classic and Multi-Purpose Workstation™ Steppers, Micro-Touch and Multi-Purpose Workstation Stabilizers, and Transportation Stands

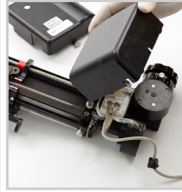

DISASSEMBLY (EX³ STEPPER ONLY)

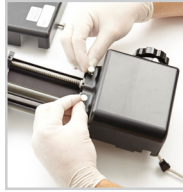

- 1** • Place on flat surface
• Loosen hold screws and remove top cover
- 2** • Rotate Stepper 180°
• Loosen hold screws and remove bottom cover
- 3** • Pull location lock finger and rotate unit to remove posterior section containing electrical components

STEPPER MANUAL CLEANING - IMMERSION OR WIPING

Immersion and wiping are both validated for cleaning. Choose either method.

Do not soak or wipe electrical components of EX³ Stepper. See posterior section in step 5.

IMMERSION

- 4** • Prepare enzymatic detergent
• Fully immerse stepper in solution and soak for 3 minutes
• After soak time, brush entire surface for 1 minute, brush all crevices and grooves
- Prepare fresh enzymatic detergent, immerse stepper, soak 3 minutes
• Remove stepper from detergent, rinse under tap water for 50-60 seconds
• Dry stepper with a clean cloth
• Air hose may be used to remove water from crevices

WIPING

- 4** • Wipe surfaces, crevices, and grooves with germicidal wipes to remove gross soil
• Wipe surfaces with germicidal wipes until visibly wet


5 EX³ posterior section only - top and bottom covers, screws and movement knob

- Wipe surfaces, crevices and grooves with germicidal wipes to remove gross soil
- Wipe surfaces with germicidal wipes until visibly wet

STEPPER DISINFECTION

- 6** • Thoroughly wet surfaces with germicidal wipes
• Ensure continuous 2 minute wet contact time
• Use additional wipes if needed for the device to remain wet
• Allow surface to air dry

ASSEMBLY (EX³ STEPPER ONLY)

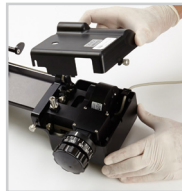

- 7** • Rotate unit to connect posterior to anterior section
• Engage lock finger and tighten interface screws
- 8** • Attach top and bottom covers, tighten hold screws and ensure USB cord is aligned with cover notch

9 STABILIZER & TRANSPORTATION STAND CLEANING & DISINFECTION

- Wipe surfaces, crevices, and grooves with germicidal wipes to remove gross soil
- Wipe surfaces with germicidal wipes until visibly wet
- Ensure continuous 2 minute wet contact time
- Use additional wipes if needed for the device to remain wet
- Allow surface to air dry



CIVCO.COM | 1.800.445.6741

COPYRIGHT © 2023. CIVCO IS REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ACCUCARE, EX3 AND MULTI-PURPOSE WORKSTATION ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. 2019R-3132 REV E