

Reprocessing EX³, Classic and Multi-Purpose Workstation™ Steppers



DISASSEMBLY (EX³ STEPPER ONLY)



- Place on flat surface
- Loosen hold screws and remove cover



- Rotate Stepper 180°
- Loosen hold screws and remove cover



- Pull location lock finger and rotate unit to remove posterior section containing electrical components

STEPPER MANUAL CLEANING

Do not soak electrical components of EX³ Stepper. See posterior section in step 3.

4 • Prepare enzymatic detergent

- Fully immerse stepper in solution and allow to soak for 3 minutes
- After soak time, brush entire surface for 1 minute, brush all crevices and grooves



5 • Dispose of detergent

- Prepare fresh enzymatic detergent, immerse stepper, soak 3 minutes
- Remove stepper from detergent, rinse under tap water for 50-60 seconds

6 • Dry stepper with clean, lint-free cloth

- Air hose may be used to remove air from crevices



7 EX³ posterior section only - top and bottom covers, screws and movement knob


- Wipe surfaces, crevice and grooves with isopropyl alcohol wipes to remove visible contaminants until visibly wet (use 7 wipes total)

DISINFECTION


- 8 • Thoroughly wet surfaces with isopropyl alcohol wipes
- Assure continuous 2 minute wet contact time
- Use additional wipes if needed for the device to remain wet
- Allow surface to air dry



ASSEMBLY (EX³ STEPPER ONLY)

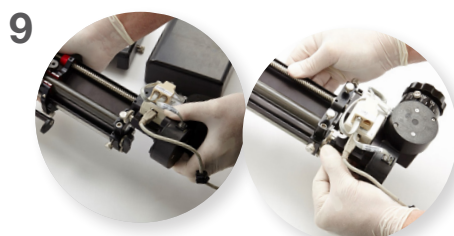

- Rotate unit to connect posterior to anterior section
- Engage lock finger and tighten interface screws

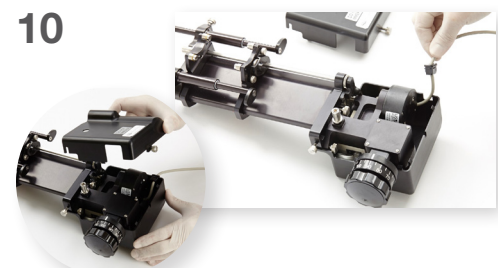

- Attach top and bottom covers, tighten hold screws and ensure USB cord is aligned with cover notch

FUNCTIONAL PRE-USE CHECKS

Prior to each use, perform the following checks to ensure optimal performance:

- Rotate longitudinal movement knob, cradle and grid rail locking knobs to ensure movement through full range of linear rotation travel
- If parts do not move freely, apply lubricant that complies with hospital protocol. Super Lube® Multi-Purpose Synthetic Grease with Syncolon® (PTFE) is recommended.

Contact CIVCO at 1.800.445.6741 for more information on enzymatic detergent, lint-free cloths and alcohol wipes to aid compliant AccuCare equipment reprocessing

CIVCO.COM | 1.800.445.6741

COPYRIGHT © 2020. CIVCO IS REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ACCUCARE, EX3 AND MULTI-PURPOSE WORKSTATION ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. 2019R-3132 REV 1

