

Reprocessing CIVCO Steppers & Stabilizers

Including EX³™, Classic and Multi-Purpose Workstation™ Steppers, Micro-Touch and Multi-Purpose Workstation Stabilizers, and Transportation Stands



DISASSEMBLY (EX³ STEPPER ONLY)

1



- Place on flat surface
- Loosen hold screws and remove top cover

2



- Rotate Stepper 180°
- Loosen hold screws and remove bottom cover

3

- Pull location lock finger and rotate unit to remove posterior section containing electrical components



STEPPER MANUAL CLEANING - IMMERSION OR WIPING

Immersion and wiping are both validated for cleaning. Choose either method.

Do not soak or wipe electrical components of EX³ Stepper. See posterior section in step 5.

IMMERSION

- Prepare enzymatic detergent
 - Fully immerse stepper in solution and allow to soak for 3 minutes
 - After soak time, brush entire surface for 1 minute, brush all crevices and grooves
- Prepare fresh enzymatic detergent, immerse stepper, soak 3 minutes
 - Remove stepper from detergent, rinse under tap water for 50-60 seconds



REF# 610-643

- Dry stepper with a clean cloth
- Air hose may be used to remove water from crevices



REF# 610-1354

WIPING

- Wipe surfaces, crevices, and grooves with germicidal wipes to remove gross soil
 - Wipes surfaces with germicidal wipes until visibly wet

5 EX³ posterior section only - top and bottom covers, screws and movement knob

- Wipe surfaces, crevices and grooves with germicidal wipes to remove gross soil
- Wipe surfaces with germicidal wipes until visibly wet

STEPPER DISINFECTION


- Thoroughly wet surfaces with germicidal wipes
 - Assure continuous 2 minute wet contact time
 - Use additional wipes if needed for the device to remain wet
 - Allow surface to air dry



REF# 610-1472

ASSEMBLY (EX³ STEPPER ONLY)

7

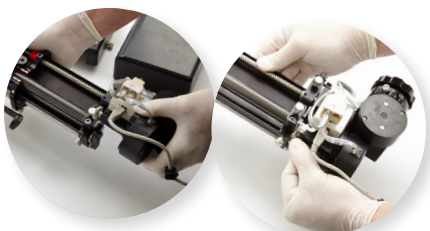

- Rotate unit to connect posterior to anterior section
- Engage lock finger and tighten interface screws

8



- Attach top and bottom covers, tighten hold screws and ensure USB cord is aligned with cover notch

9

STABILIZER & TRANSPORTATION STAND CLEANING & DISINFECTION

- Wipe surfaces, crevices, and grooves with germicidal wipes to remove gross soil
- Wipes surfaces with germicidal wipes until visibly wet
- Assure continuous 2 minute wet contact time
- Use additional wipes if needed for the device to remain wet
- Allow surface to air dry



Contact CIVCO at 1.800.445.6741 for more information on enzymatic detergent, drying cloths and germicidal wipes to aid in compliant AccuCare equipment reprocessing

CIVCO.COM | 1.800.445.6741

COPYRIGHT © 2021. CIVCO IS REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ACCUCARE, EX3 AND MULTI-PURPOSE WORKSTATION ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. 2019R-3132 REV 3

