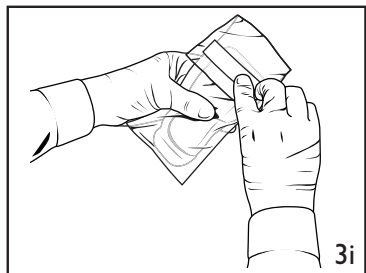
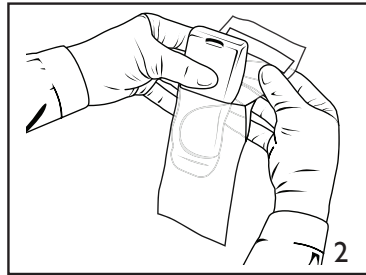
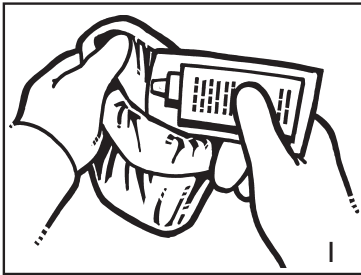
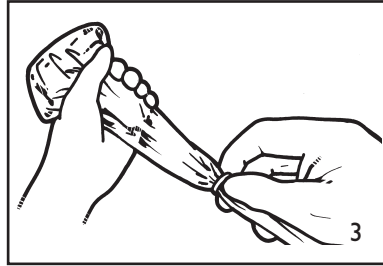
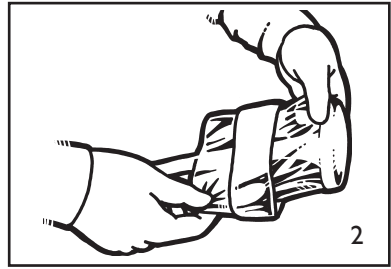
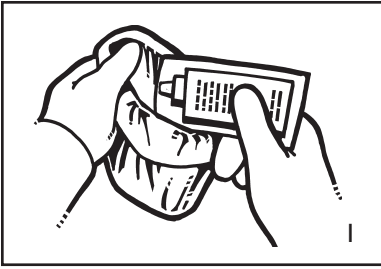
Latex-Free Transducer Covers

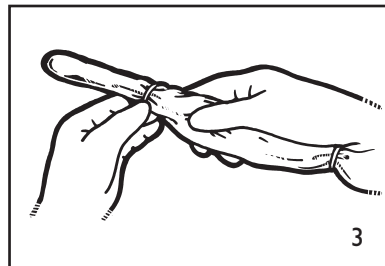
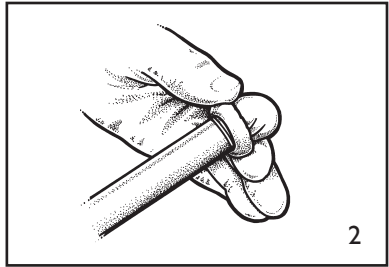
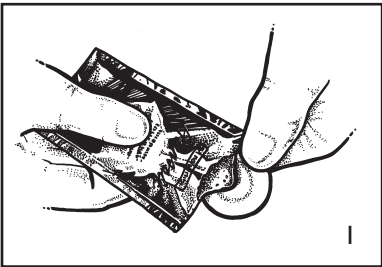
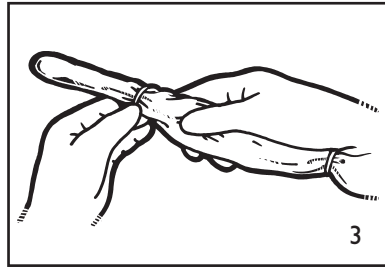
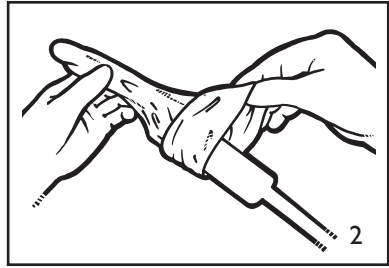
CIV-Flex™ • NeoGuard™ • iVAS™ • Poly • Surgi™ • Surgi-Tip™

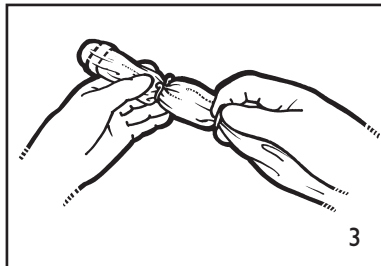
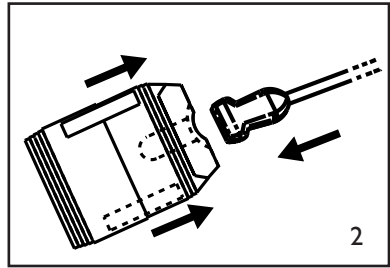
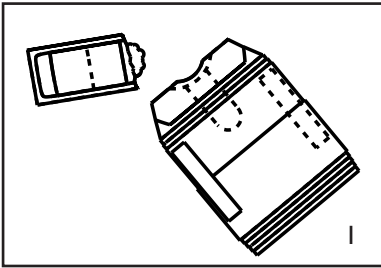
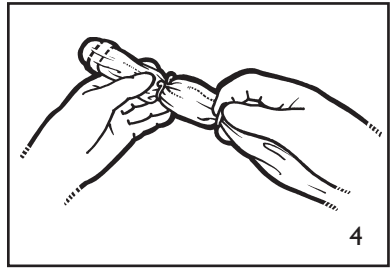
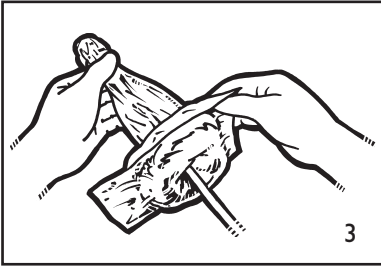
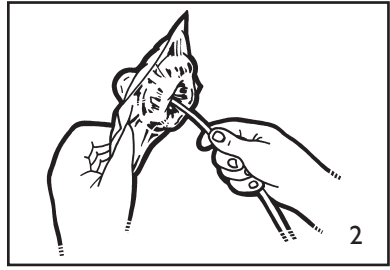
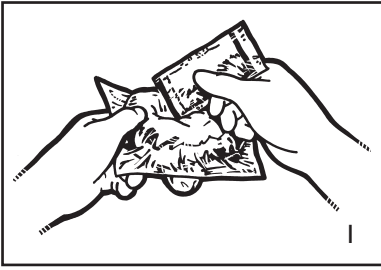
REFERENCE GUIDE

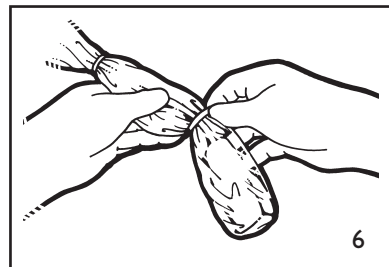
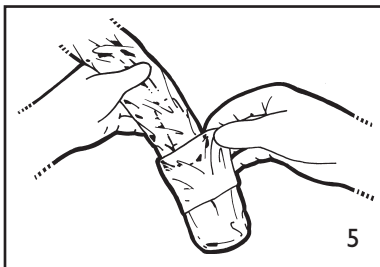
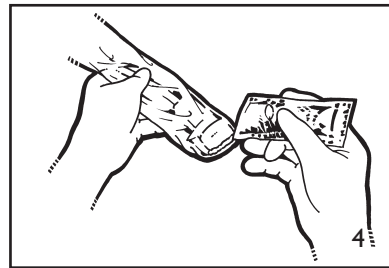
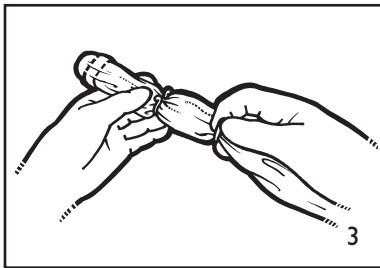
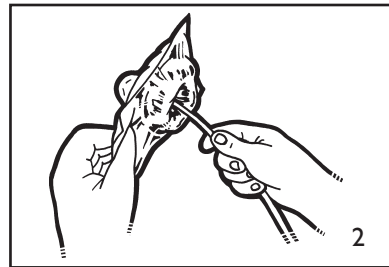
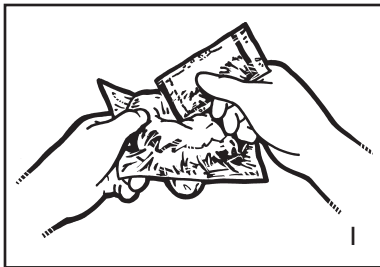
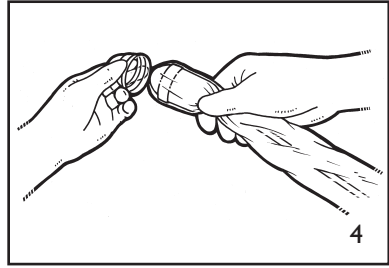
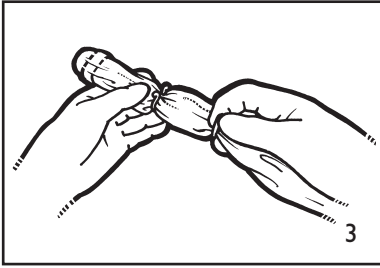
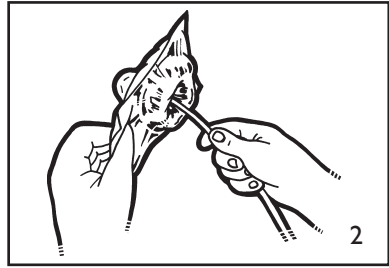
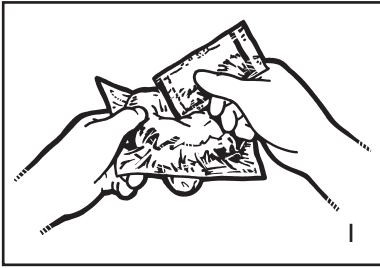
CE
0120

INTENDED USE

- The cover allows use of the transducer in scanning and needle guided procedures for body surface, endocavity and intra-operative diagnostic ultrasound, while helping to prevent transfer of microorganisms, body fluids and particulate material to the patient and health care worker during reuse of the transducer.

WARNING

- *In the USA, federal law restricts device to sale or use by or on order of a physician.*
- *Before use, you should be trained in ultrasonography. For instructions on the use of your transducer, see your system's manual.*
- *Dispose of single-use components as infectious waste.*
- *See your system's manual for reprocessing transducer between use.*
- *Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.*

Covering the Transducer and Bracket

WARNING

- *Use only water-soluble agents or gels. Petroleum or mineral oil based materials may harm cover.*
- *Disposable components are single-use only. Do not use if expiration date has passed.*
- *For disposable components labeled sterile, do not use if integrity of package is violated.*
- *For illustration purposes only, transducer may be shown without a transducer cover. Always place a cover over transducer to protect patients and users from cross-contamination.*

1. Place an appropriate amount of gel inside cover and/or on transducer face. Poor imaging may result if no gel is used.
2. Insert transducer into cover making sure to use proper sterile technique. Pull cover tightly over transducer face to remove wrinkles and air bubbles, taking care to avoid puncturing cover.
3. Secure with enclosed bands or remove adhesive liner and fold cover over to enclose.
4. Inspect cover to ensure there are no holes or tears.

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Corporate Headquarters: 102 First Street South, Kalona, IA 52247-9589 USA
319.248.6757, 800.445.6741 | fax: 319.248.6660, 877.329.2482
www.civco.com

COPYRIGHT © 2013 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. CIVFLEX, IVAS AND NEOGUARD ARE TRADEMARKS OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA.

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MPS GmbH