POSITIONING ARM COVER

REFERENCE GUIDE

CIVCO *Assist*[™]

INTENDED USE

Positioning arm cover helps to prevent transfer of microorganisms, body fluids and particulate material to patient and health care worker.

CAUTION

Federal (United States) law restricts this device to sale by or on the order of a physician.

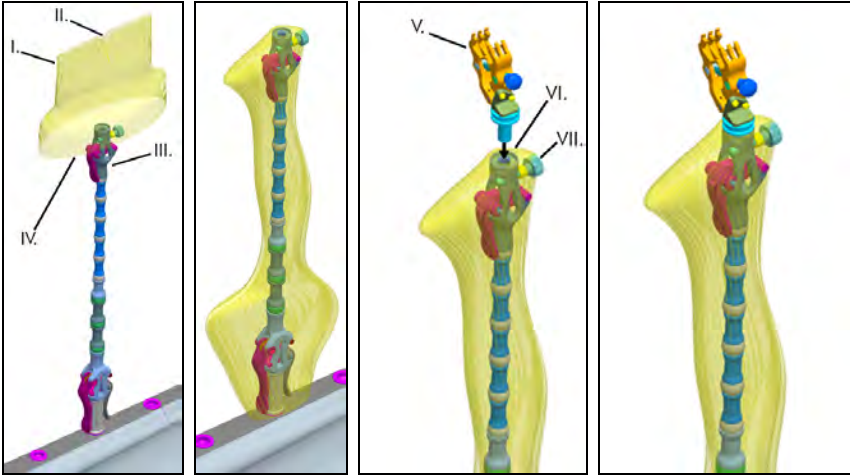
WARNING:

- *Users of this product have an obligation and responsibility to provide the highest degree of infection control to patients, co-workers and themselves. To avoid cross-contamination, follow all infection control policies established for your office, department or hospital as they apply to personnel and equipment.*
- *Before use, you should be trained in CT and ultrasound. For instructions on the use of CIVCO Assist, see system's manual.*
- *See system's manual for reprocessing positioning arm and attachments.*
- *Dispose of single-use components as infectious waste.*
- *Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed.*
- *Replace cover if attachment is removed after it has been inserted into Quick-Mount.*
- *Take care to avoid damaging cover during use.*
- *Do not reuse, reprocess or resterilize single-use device. Reuse, reprocessing or resterilization may create a risk of contamination of the device, cause patient infection or cross-infection.*

USING POSITIONING ARM COVER

NOTE: Refer to specific attachment instruction to determine if attachment is installed under or over cover.

1. Orient positioning arm vertically and lock exacting lever.
2. Place cover using proper sterile technique.
3. Leave excess cover material to ensure levers can be locked and unlocked easily without damaging cover. Secure with bands or tape.
4. If applicable, insert attachment through Quick-Mount attachment opening in cover and into Quick-Mount. Avoid pushing cover into Quick-Mount.

- I. Positioning Arm Cover
- II. Quick-Mount™ attachment opening
- III. Positioning Arm
- IV. Exacting Lever
- V. Attachment
- VI. Quick-Mount
- VII. Quick-Mount Lock

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COPYRIGHT © 2022 ALL RIGHTS RESERVED. CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. ALL OTHER TRADEMARKS ARE PROPERTY OF THEIR RESPECTIVE OWNERS. PRINTED IN USA. ALL PRODUCTS MAY NOT BE LICENSED IN ACCORDANCE WITH CANADIAN LAW.
CIVCO Medical Instruments Co., Inc.